



Utilizing the LID57X Toolbox Program with the LID573/LID575

The LID57X Toolbox Program can be used in conjunction with the LID 573/LID 575 readers to record Trovan's unique transponder identification along with any given user/customer identification, allocated to the individual animal. The reader will store this information and/or upload it to a USB or wirelessly (Bluetooth) connected computer. In addition, these identification details can be easily exported to an Excel spreadsheet for further storage and archiving. The instructions given below provide a step-by-step guide on how to utilize the 57X Toolbox Program.


1. Click on the link www.microchipid.us/software to open/start the program.
2. The start screen will appear
3. Select Soft57x up to V6xx

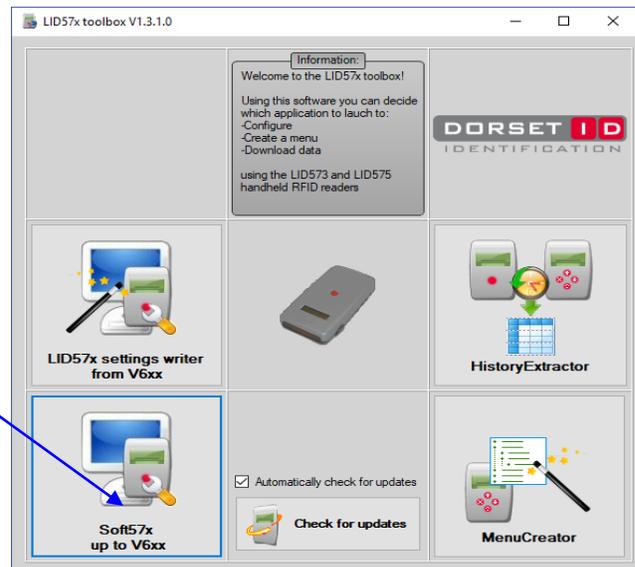

4. Connect the LID 573 to the USB port on your PC
5. USB connect will automatically be displayed

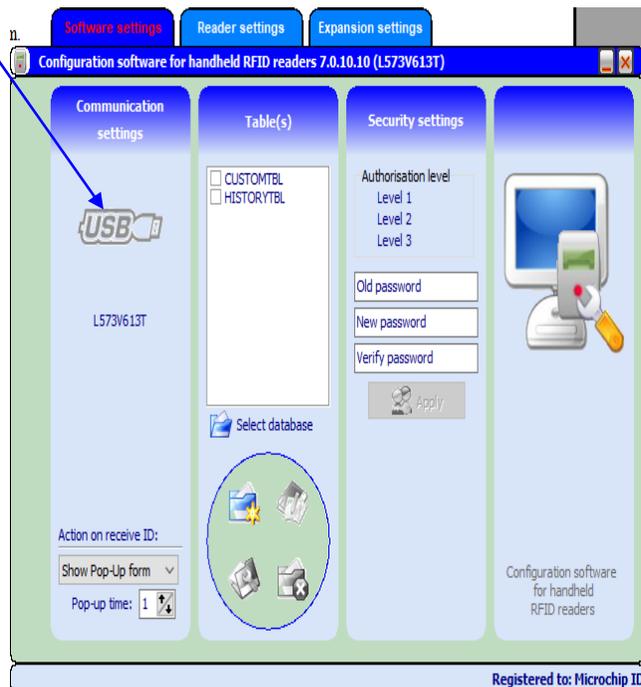


6. Click on the “Readers Setting Tab”
7. You can select the various options you want the reader to have
8. The default options which should be checked are marked with red arrows

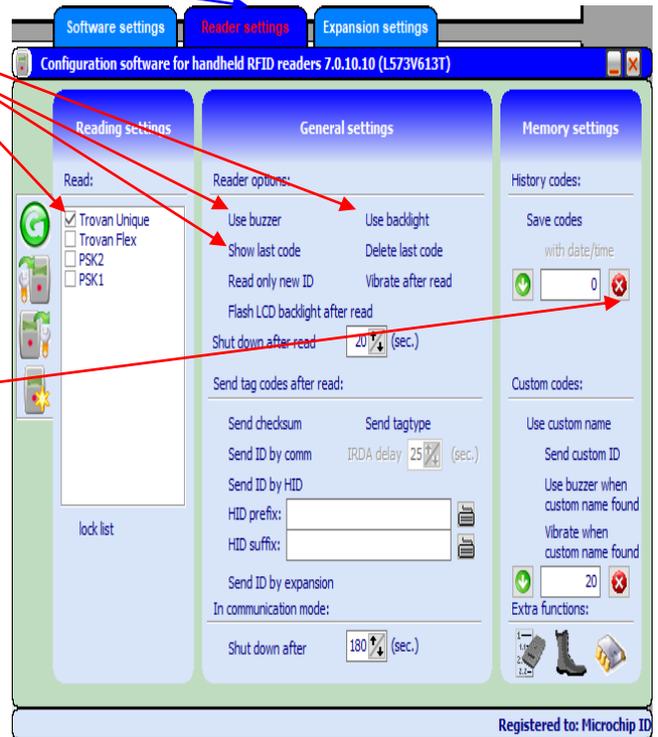


9. Clear the memory from the reader by clicking the Red X


10. Un-check the “Send ID tagtype” box
11. Uncheck “Send checksum” box
12. Check the “Send ID by Comm” box if you are connected to a data storage device (computer)
13. Settings to the reader is automatic
14. Scan all the transponders you wish to have mapped into the database.
15. Scan and “Save Codes” without the, date and time, you want to have in your user table
16. Reconnect the scanner to the cable ????
17. Select download history codes

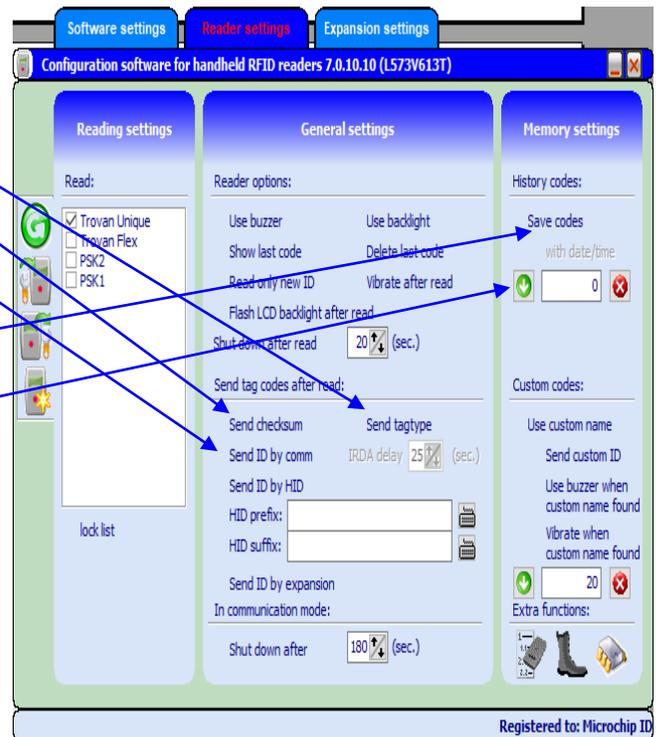

18. Select "CUSTOMETBLE" and "Select"

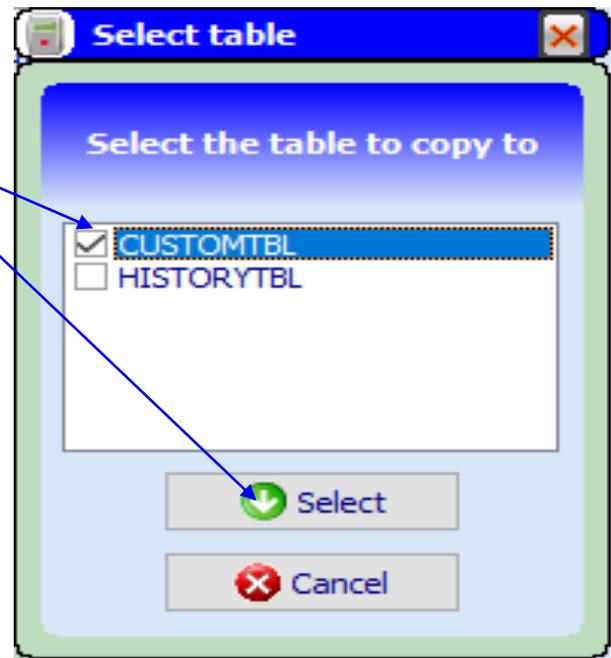

19. Right Click and open "auto increment tool"

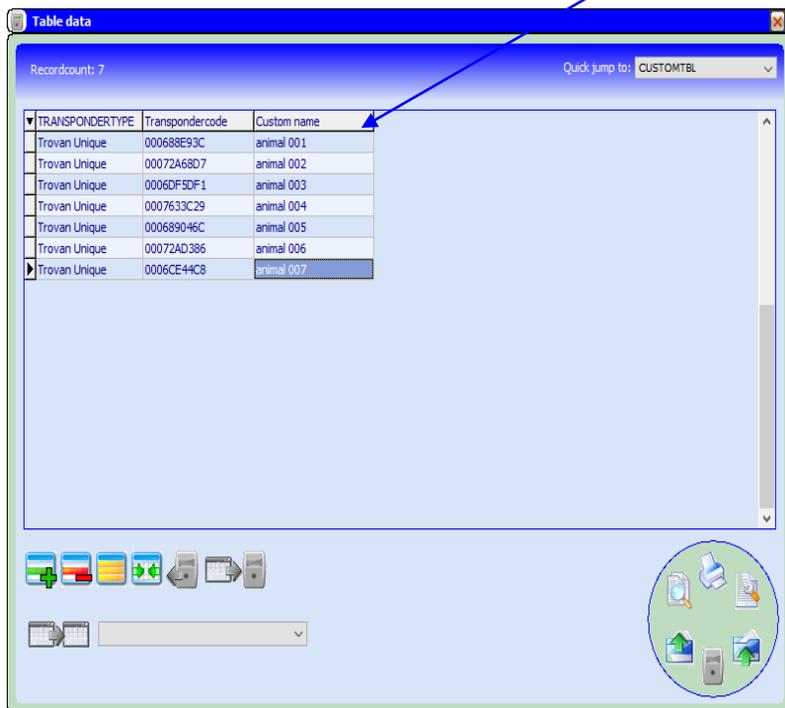

19. Add text, Increment Value, From current or first record, and number of records,

20. Finish off by clicking here

Auto increment for: [FIELDNAME]

TotalSizeLabel

Increment properties

Add text: animal

Start value: 1

Increment value: 1

Fixed length at: 3

Example: animal 001

Auto increment:

from current record

from first record

Number of records: 7


Table data

Recordcount: 7 Quick jump to: CUSTOMTBL


TRANSPONDERTYPE	Transpondercode	Custom name
Trovan Unique	000688E93C	
Trovan Unique	00072A68D7	
Trovan Unique	0006DF5DF1	
Trovan Unique	0007633C29	
Trovan Unique	000689046C	
Trovan Unique	00072AD386	
Trovan Unique	0006CE44C8	

21. Upload customer codes by clicking here


22. Select "Yes"

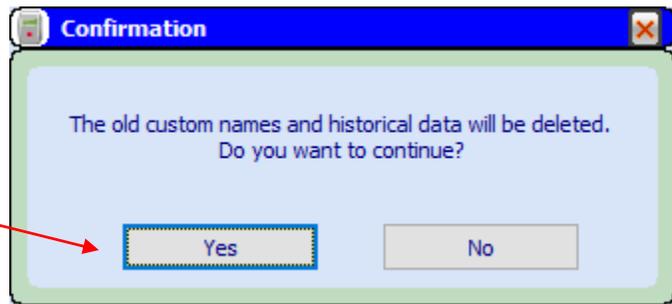


23. Select Export to a file, this will export to an excel file

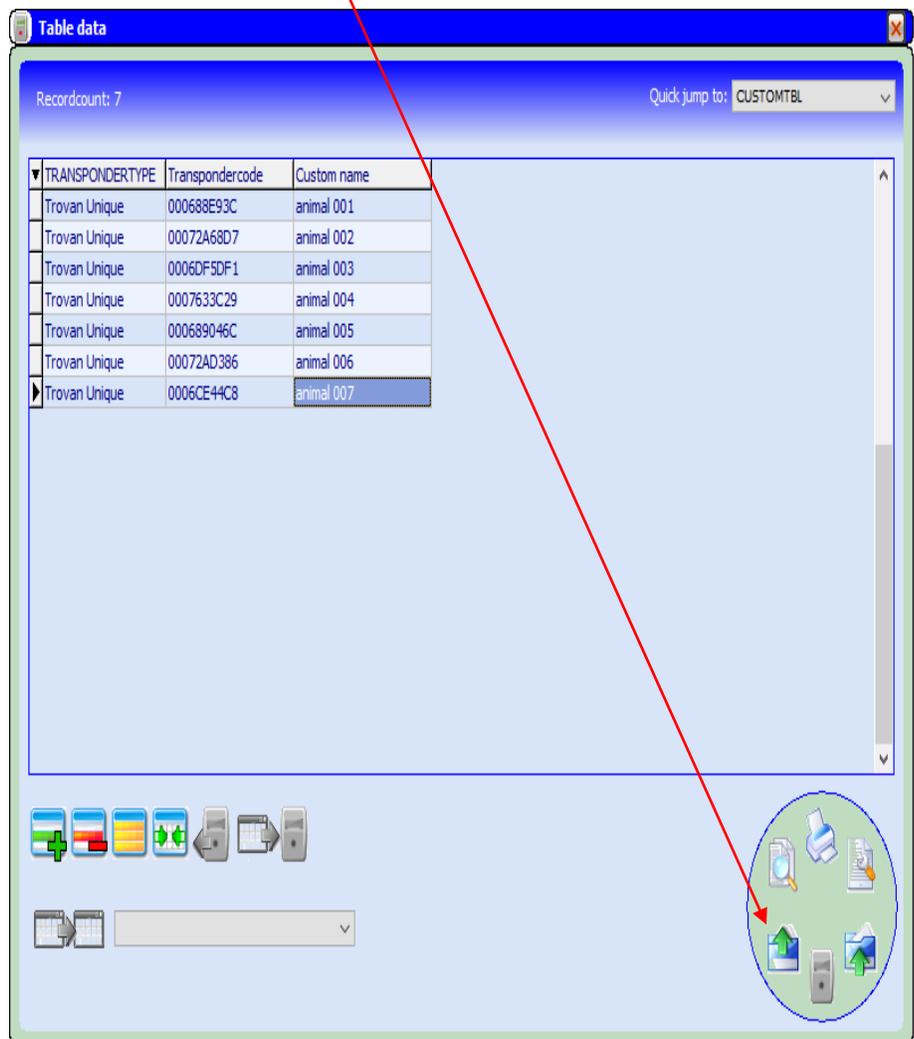

24. Select all fields and click on "Next"

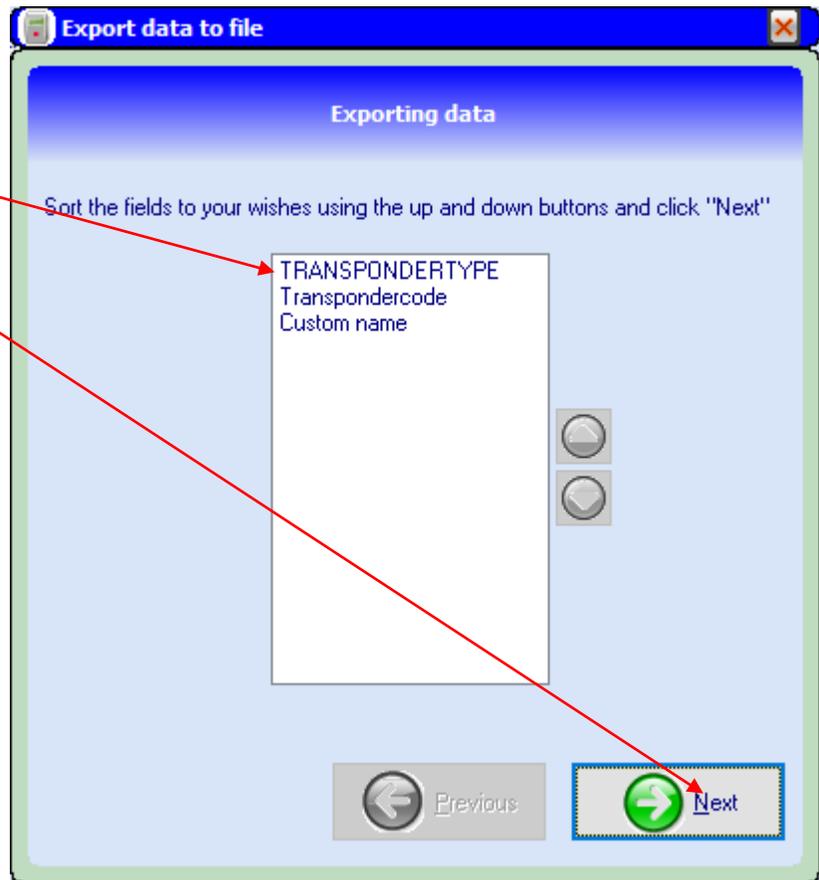

25. Click on "Next"

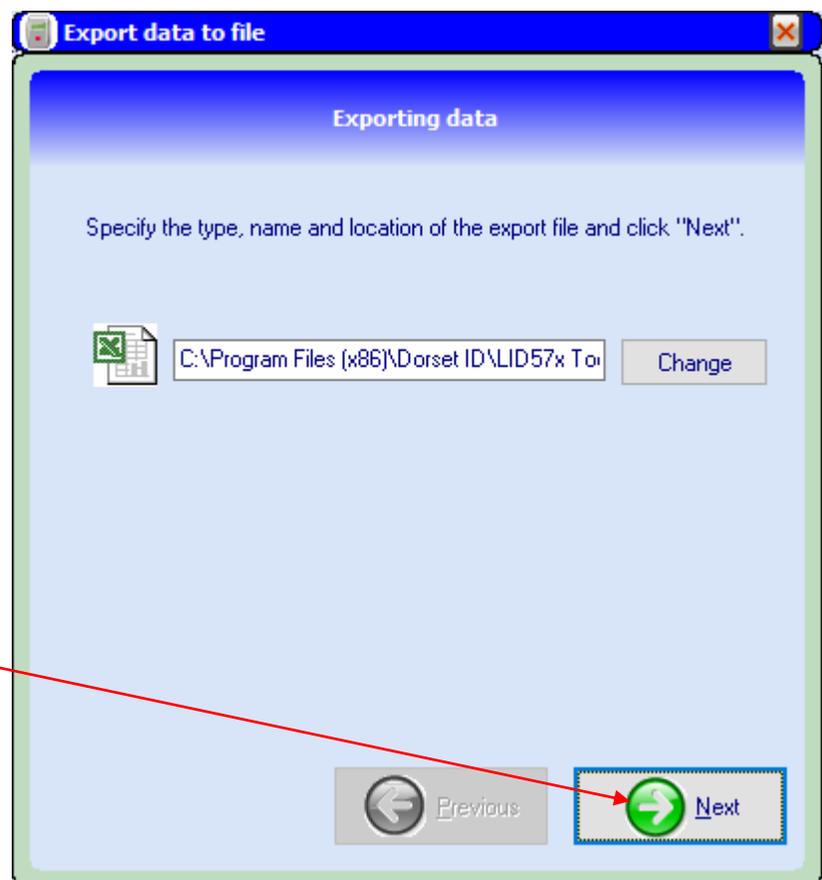



26. Click on "Finish"