



Download the 57x Toolbox Software

<https://www.microchipid.us/support>



LID 57x Toolbox

LID57x toolboxV1.4.8.0

Information  

Welcome to the LID57x toolbox!

Using this software you can decide which application to launch to:

- Configure your reader;
- Download data from your reader.


Compatible with the LID573 and LID575 readers.

DORSET ID
IDENTIFICATION


Visit our website @ www.dorset.nu

LID57x Toolbox

No reader connected



Please connect a LID57x handheld RFID reader to your computer.

 Check for updates

Automatically check for updates



LID 57x Toolbox

LID57x toolboxV1.4.8.0

Information  

Welcome to the LID57x toolbox!

Using this software you can decide which application to launch to:

- Configure your reader;
- Download data from your reader.

Compatible with the LID573 and LID575 readers.

DORSET ID
IDENTIFICATION

Visit our website @ www.dorset.nu

LID57x Toolbox


Reader version L573V721t

| | | |
|---|---|--|
|  LID57x Wizard |  |  History extractor |
|  LID57x Bootloader | | Custom Names  |


 Check for updates Automatically check for updates




LID 57x Toolbox

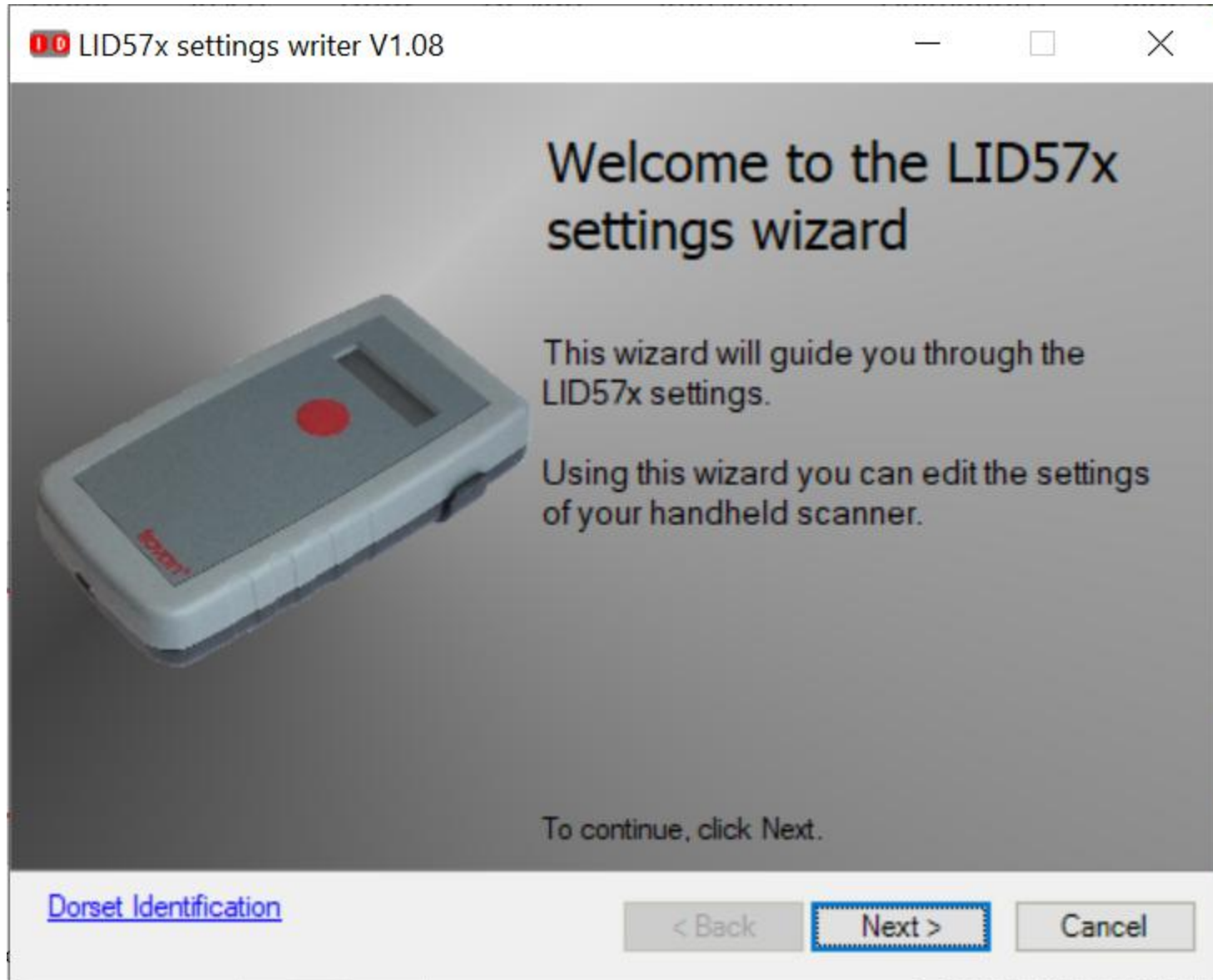

LID 57x Toolbox

 LID57x settings writer V1.08

Select transponders to read
Please select the transponder types the reader should read



Current firmware version: L573V721t

- Trovan unique
- Trovan Flex
- PSK1
- PSK2


To increase reading performance
only select required transponder types.

[Dorset Identification](#)

< Back **Next >** Cancel



LID 57x Toolbox

 LID57x settings writer V1.08



General settings

Select general settings



- | | |
|---|---|
| <input checked="" type="checkbox"/> Use buzzer | <input checked="" type="checkbox"/> Use backlight |
| <input checked="" type="checkbox"/> Show last code | <input checked="" type="checkbox"/> Delete last code on lcd |
| <input type="checkbox"/> Read only new ID | <input type="checkbox"/> Vibrate after read |
| <input type="checkbox"/> Flash LCD backlight after read | <input checked="" type="checkbox"/> Use custom names |
| <input type="checkbox"/> Lock readlist and menu | |

Custom names options

- Send custom names
- Use buzzer custom found
- Vibrate custom found

Shutdown time after read (s):


[Dorset Identification](#)

< Back **Next >** Cancel



LID 57x Toolbox

MICROCHIP 

 LID57x settings writer V1.08



Save settings

Save tag code to reader memory options



Save codes

Memory size 512 Kbit

with date/time

Number of codes without date/time: 7281

Number of codes with date/time: 4369

Warning!

Changing the save options will delete all present data in the LID57x.

[Dorset Identification](#)


< Back

Next >

Cancel



LID 57x Toolbox

 LID57x settings writer V1.08



Communication settings

Set communication options



Send id by USB

Send ID by USB

USB Comport

USB Keyboard

USB Keyboard

Prefix

ENTER

TAB

Suffix

ENTER

TAB

Send id by Bluetooth

Send ID by Bluetooth

Keep connected

Bootup connect, after sent disconnect

Always connected

Connect when tag found, after sent disconnect

Send tagtype

Send checksum

Shutdown time after communication:

180



[Dorset Identification](#)

< Back


Next >

Cancel



LID 57x Toolbox

MICROCHIP I 

 LID57x settings writer V1.08



Completed

Ready for use



The reader is ready for use.

Please disconnect the reader.

[Dorset Identification](#)


< Back

Finish


Cancel



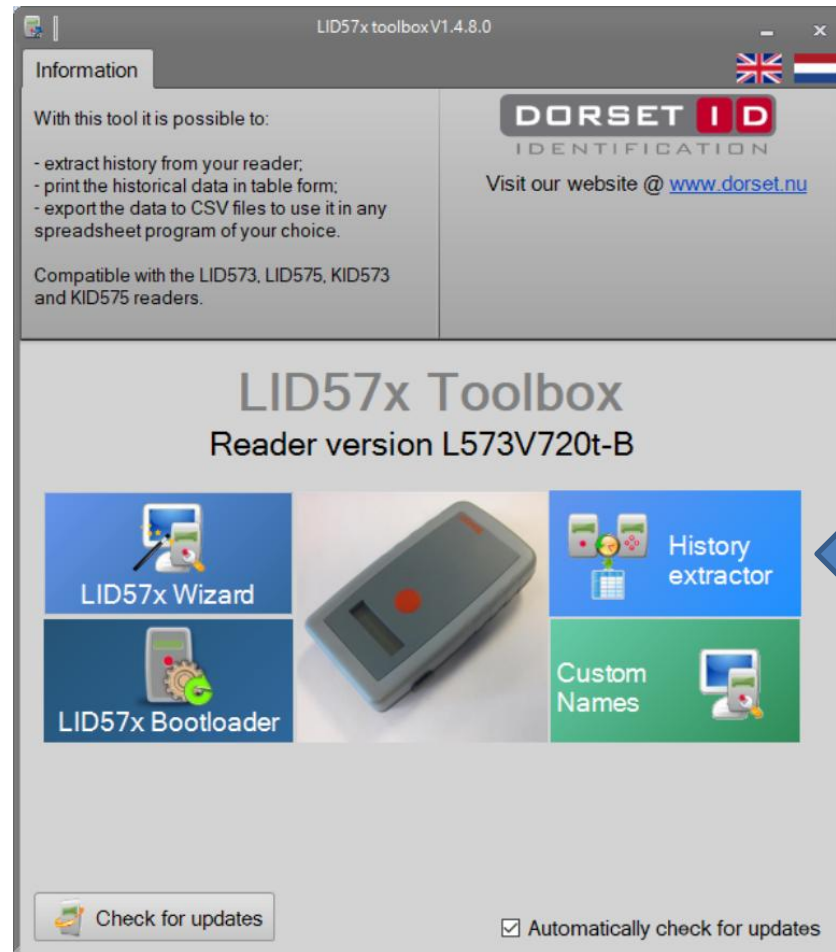

History Extractor

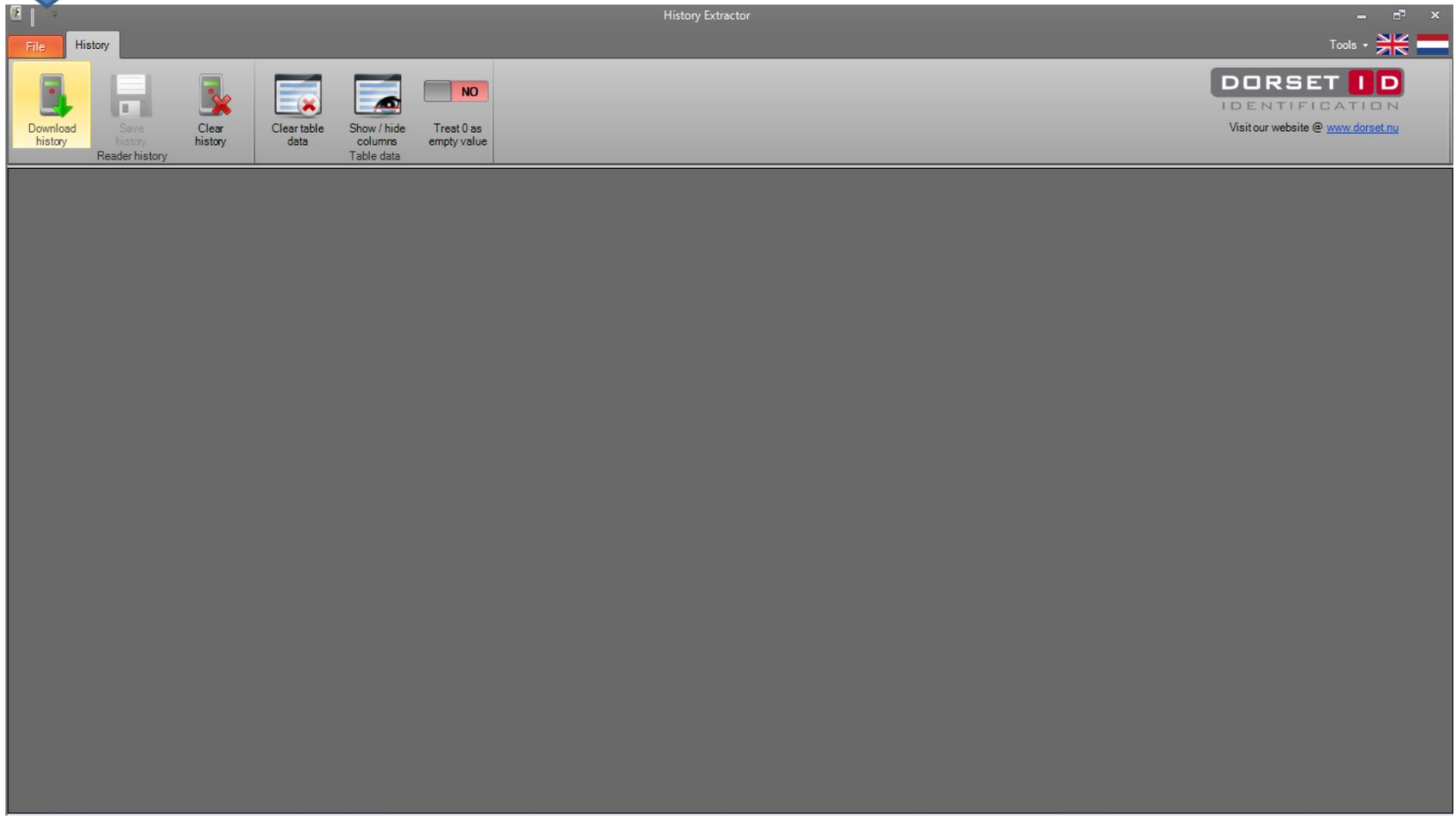

Download saved readings



History Extractor


File History Tools  

DORSET ID
IDENTIFICATION
Visit our website @ www.dorset.nu

Download history Save history Reader history Clear history Clear table data Show / hide columns Table data Treat 0 as empty value



Downloaded Readings

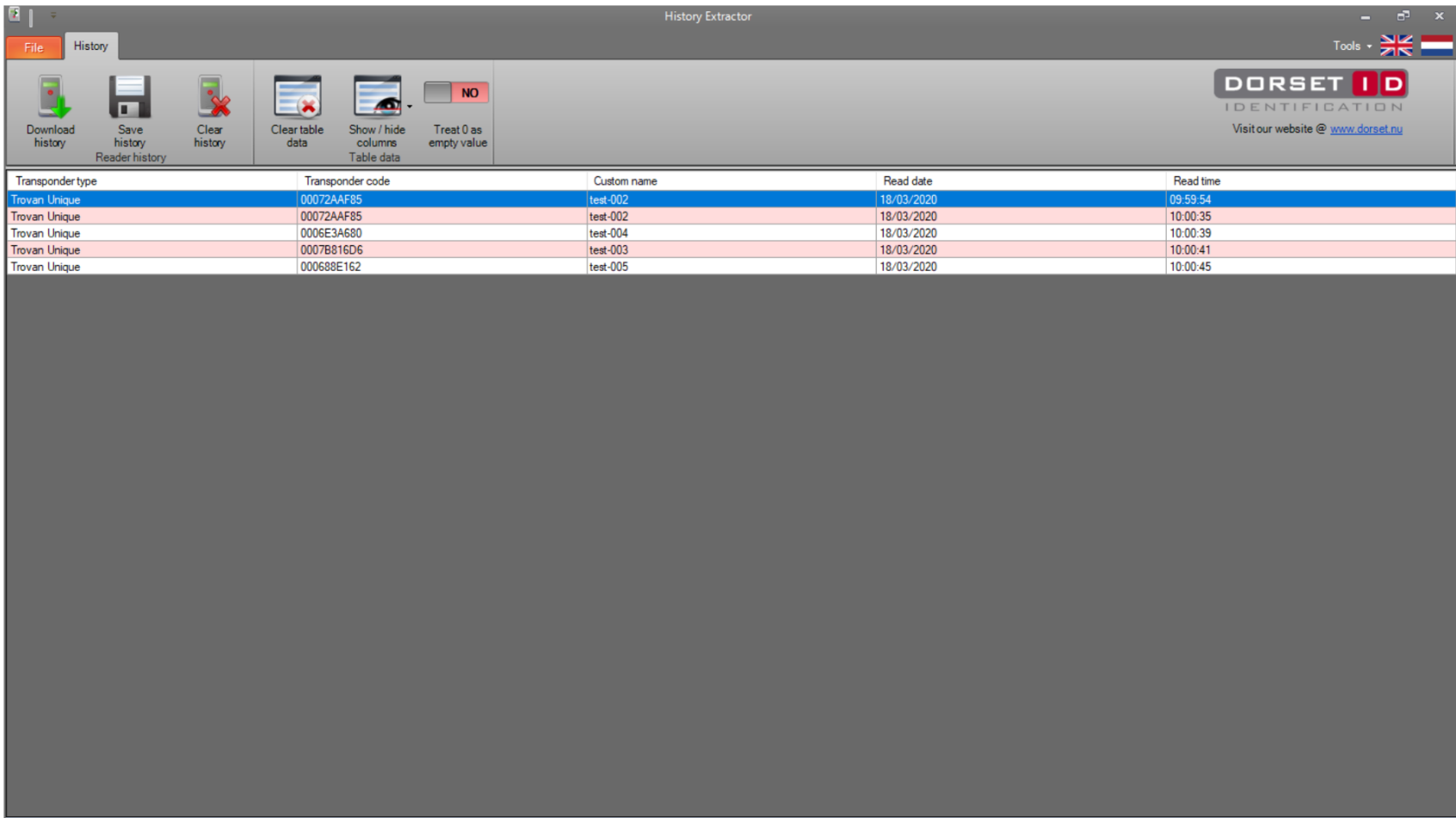


The screenshot shows the 'History Extractor' application window. The title bar reads 'History Extractor'. The interface includes a menu bar with 'File' and 'History' options, and a toolbar with icons for 'Download history', 'Save history Reader history', 'Clear history', 'Clear table data', 'Show / hide columns Table data', and 'Treat 0 as empty value'. The 'DORSET ID IDENTIFICATION' logo is visible in the top right corner, along with the text 'Visit our website @ www.dorset.nu'. The main area displays a table with the following data:

| Transponder type | Transponder code | Custom name | Read date | Read time |
|------------------|------------------|-------------|------------|-----------|
| Trovan Unique | 00072AAF85 | test-002 | 18/03/2020 | 09:59:54 |
| Trovan Unique | 00072AAF85 | test-002 | 18/03/2020 | 10:00:35 |
| Trovan Unique | 0006E3A680 | test-004 | 18/03/2020 | 10:00:39 |
| Trovan Unique | 0007B816D6 | test-003 | 18/03/2020 | 10:00:41 |
| Trovan Unique | 000688E162 | test-005 | 18/03/2020 | 10:00:45 |



Save file to your computer

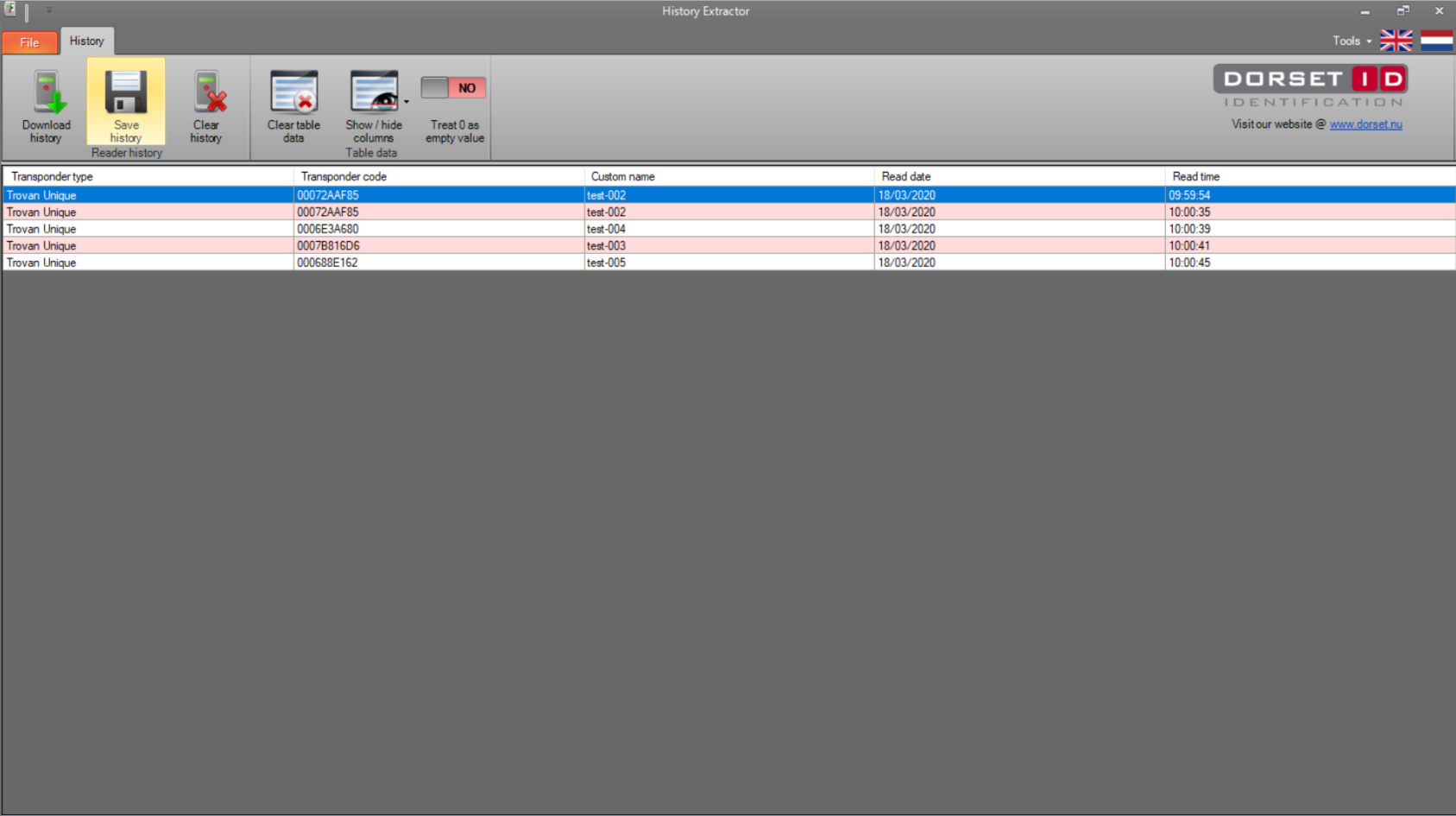


The screenshot shows the 'History Extractor' application window. The title bar reads 'History Extractor'. The interface includes a menu bar with 'File' and 'History', and a toolbar with various icons. The 'Save history' icon, which shows a floppy disk, is highlighted in yellow. Other icons include 'Download history', 'Clear history', 'Clear table data', 'Show / hide columns Table data', and 'Treat 0 as empty value'. In the top right corner, there is a 'Tools' dropdown menu, flags for the United Kingdom and the Netherlands, and the 'DORSET ID IDENTIFICATION' logo with the website address www.dorset.nu.


| Transponder type | Transponder code | Custom name | Read date | Read time |
|------------------|------------------|-------------|------------|-----------|
| Trovan Unique | 00072AAF85 | test-002 | 18/03/2020 | 09:59:54 |
| Trovan Unique | 00072AAF85 | test-002 | 18/03/2020 | 10:00:35 |
| Trovan Unique | 0006E3A680 | test-004 | 18/03/2020 | 10:00:39 |
| Trovan Unique | 0007B816D6 | test-003 | 18/03/2020 | 10:00:41 |
| Trovan Unique | 000688E162 | test-005 | 18/03/2020 | 10:00:45 |



Select folder and create a name

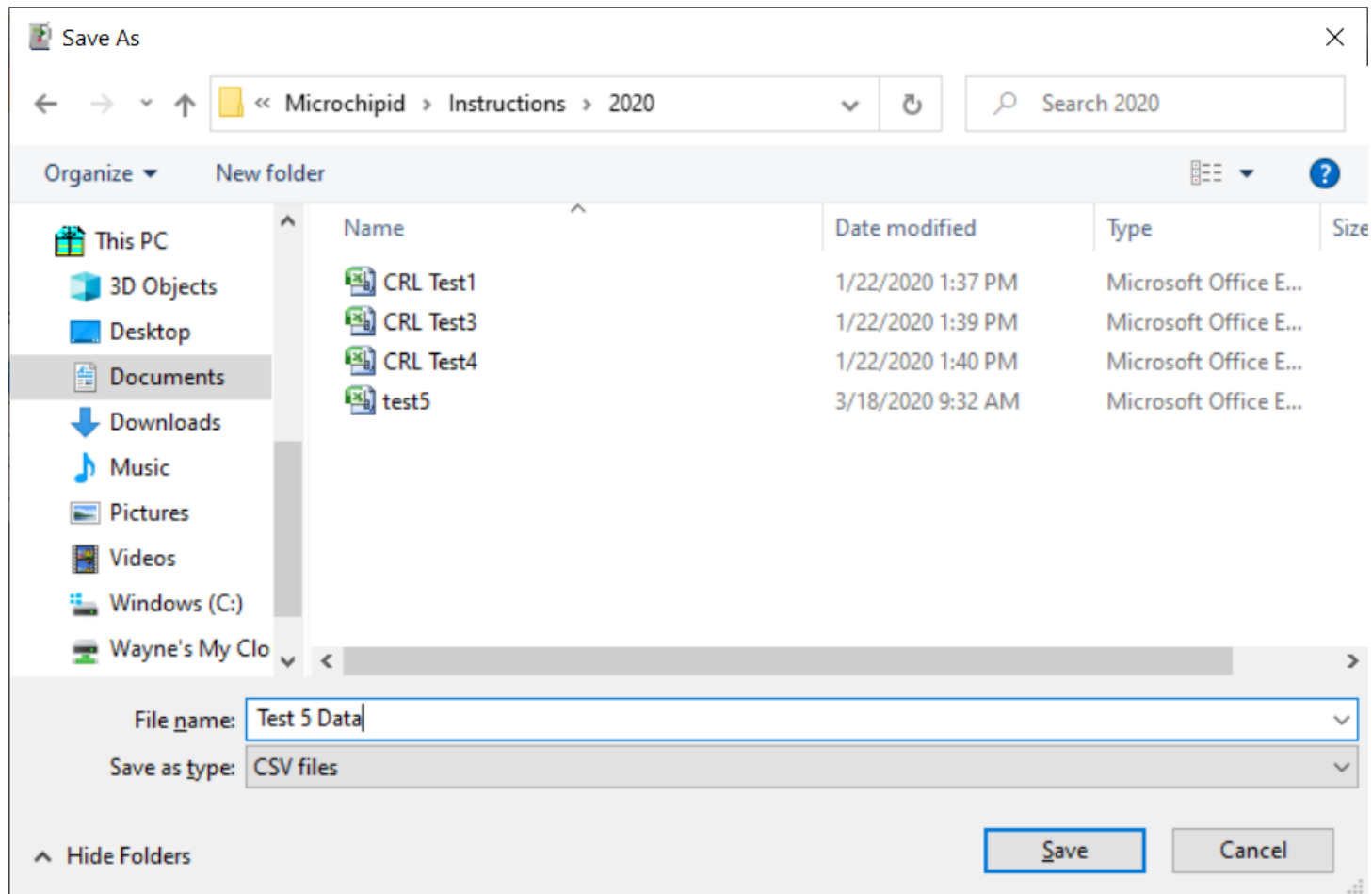

Click on OK

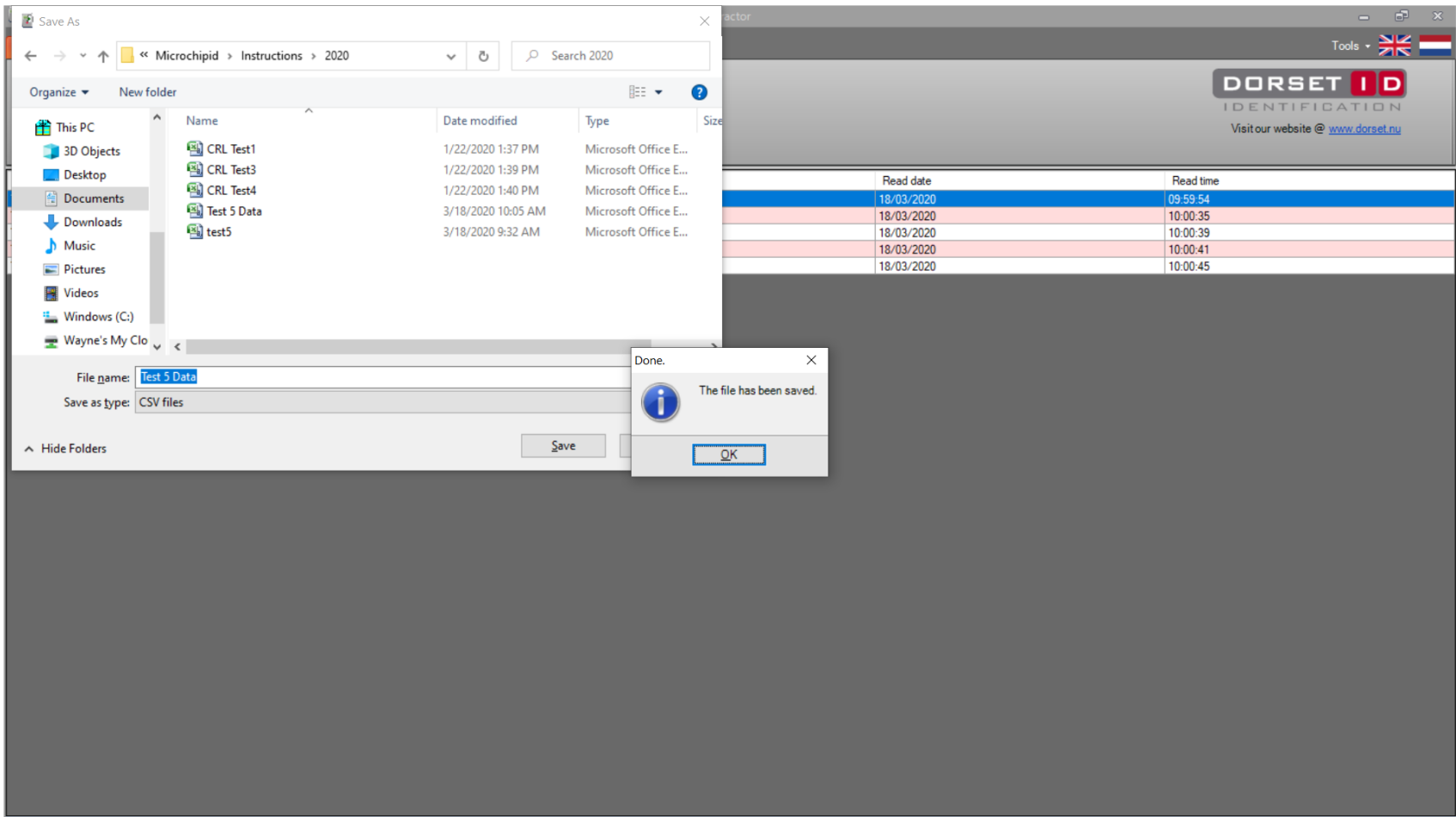

The screenshot displays a software interface with a 'Save As' dialog box open. The dialog box shows the file name 'Test 5 Data' and the save type 'CSV files'. A 'Done' notification box is overlaid on the dialog, stating 'The file has been saved.' with an 'OK' button. In the background, a table with two columns, 'Read date' and 'Read time', is visible. The table contains five rows of data, with the first row highlighted in blue and the others in pink.

| Read date | Read time |
|------------|-----------|
| 18/03/2020 | 09:59:54 |
| 18/03/2020 | 10:00:35 |
| 18/03/2020 | 10:00:39 |
| 18/03/2020 | 10:00:41 |
| 18/03/2020 | 10:00:45 |



LID 57x Toolbox

LID57x toolbox V1.4.8.0

Information

Welcome to the LID57x toolbox!

Using this software you can decide which application to launch to:

- Configure your reader.
- Download data from your reader.

Compatible with the LID573 and LID575 readers.

DORSET ID
IDENTIFICATION

Visit our website @ www.dorset.nu

LID57x Toolbox
Reader version L573V721t

LID57x Wizard

LID57x Bootloader

History extractor

Custom Names

Check for updates

Automatically check for updates



Click on connect

Custom Name Generator

File Actions Display

Connect Disconnect New file Import from file Append from file Download from reader Upload to reader Clear reader custom names Start batch Abort batch Total record count: 0

| Transponder code | Transponder type | Custom name |
|------------------|------------------|-------------|
|------------------|------------------|-------------|


Not connected

Autoscan disabled

Save file




Click on Start Batch

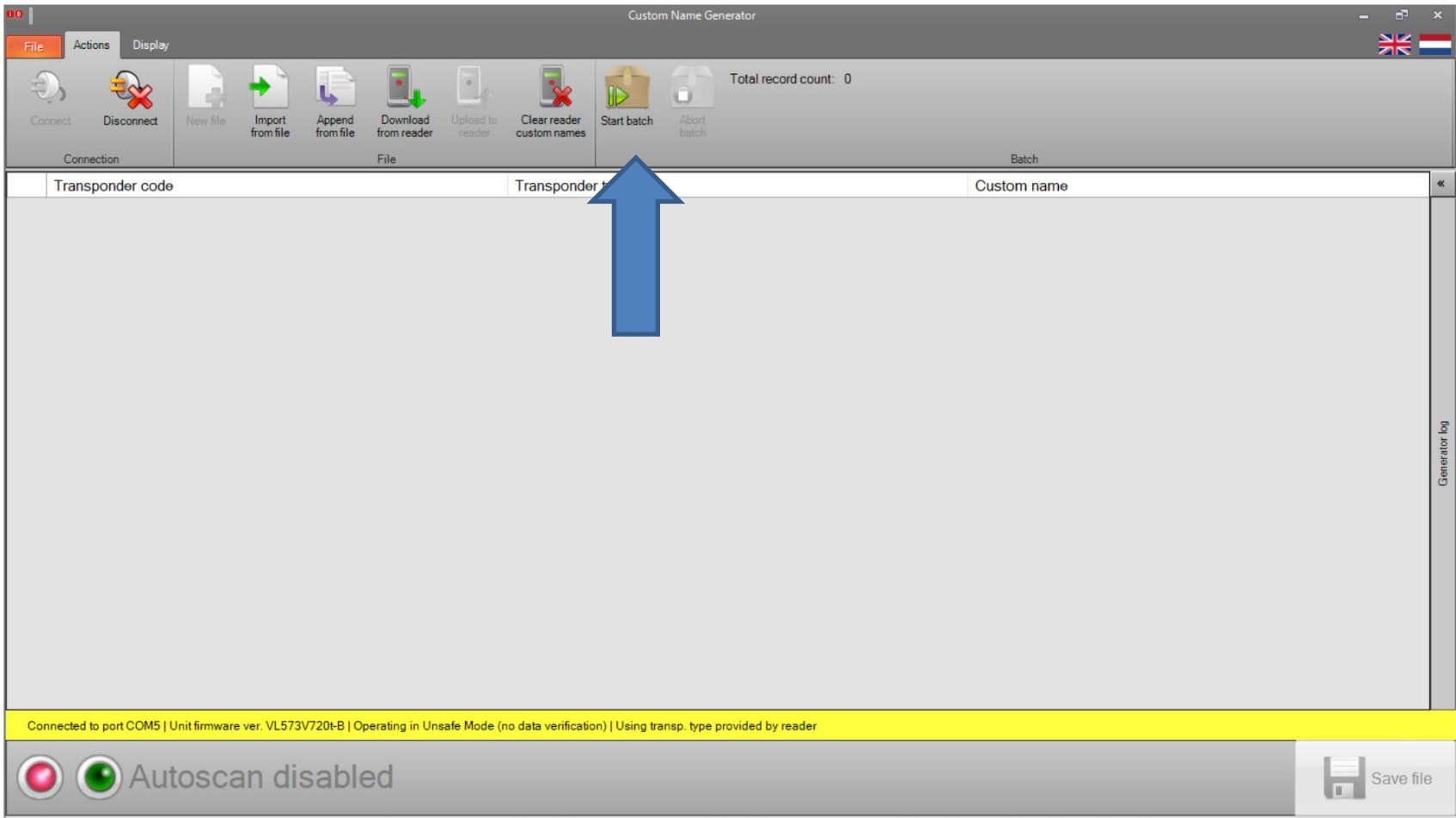

Start batch screen

Reader custom name generator - Start Batch

 **Start batch**
Add custom names by the numbers

Custom name pattern settings

Custom name prefix:



Start value: Use fixed length:

Step size: Fixed length:

0

Batch settings


Number of records in this batch:

 Cancel  Start batch



Add Prefix and enter Parameters

Reader custom name generator - Start Batch

 **Start batch**
Add custom names by the numbers

Custom name pattern settings

Custom name prefix:



Start value: Use fixed length:

Step size: Fixed length:

test-001


Batch settings

Number of records in this batch:

 Cancel  Start batch



Screen after 5 Transponders were entered

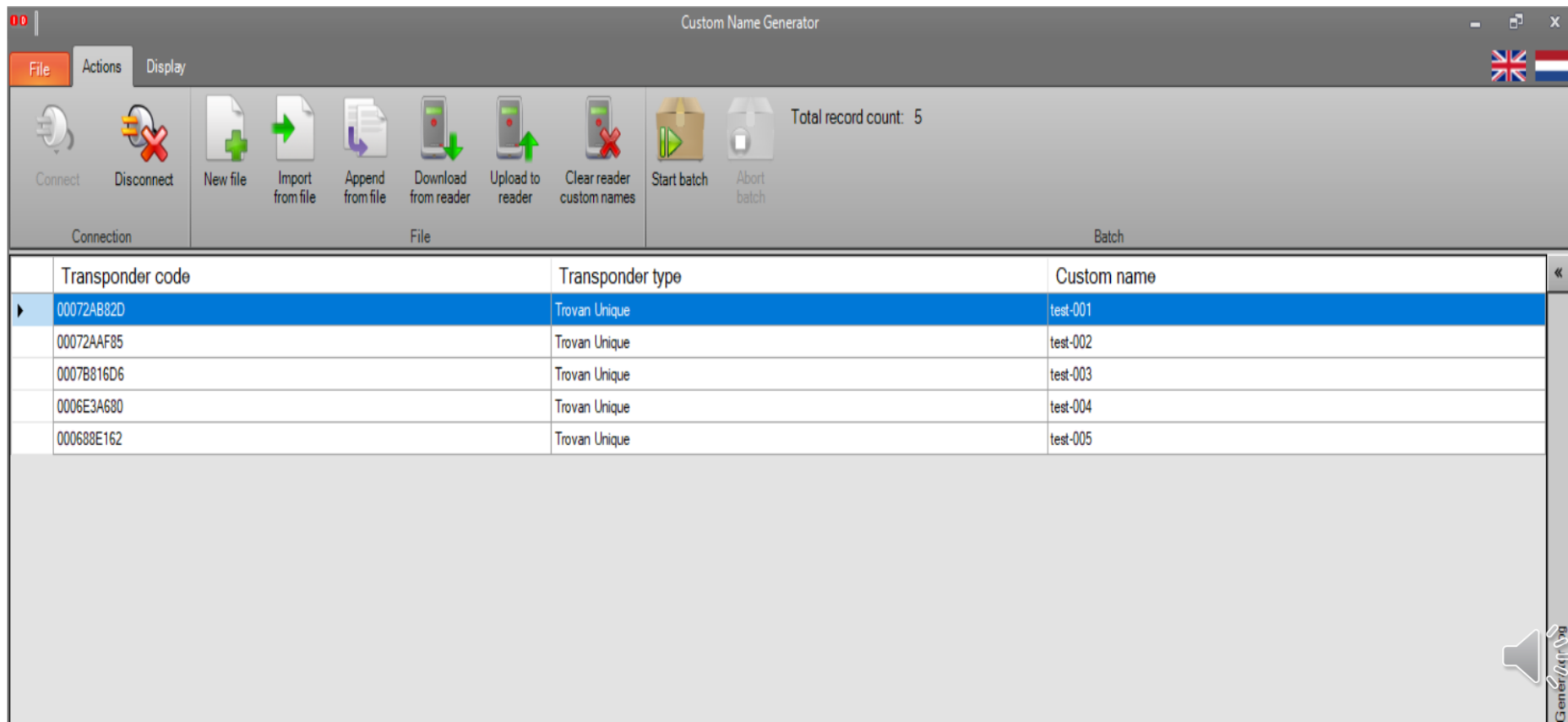


The screenshot shows the 'Custom Name Generator' software interface. The window title is 'Custom Name Generator'. The interface includes a menu bar with 'File', 'Actions', and 'Display'. Below the menu bar is a toolbar with icons for 'Connect', 'Disconnect', 'New file', 'Import from file', 'Append from file', 'Download from reader', 'Upload to reader', 'Clear reader custom names', 'Start batch', and 'Abort batch'. The 'Total record count: 5' is displayed on the right side of the toolbar. The main area contains a table with the following data:


| Transponder code | Transponder type | Custom name |
|------------------|------------------|-------------|
| 00072AB82D | Trovan Unique | test-001 |
| 00072AAF85 | Trovan Unique | test-002 |
| 0007B816D6 | Trovan Unique | test-003 |
| 0006E3A680 | Trovan Unique | test-004 |
| 000688E162 | Trovan Unique | test-005 |

The first row is highlighted in blue. The bottom right corner of the window shows a speaker icon and the text 'General'.

Save file to computer

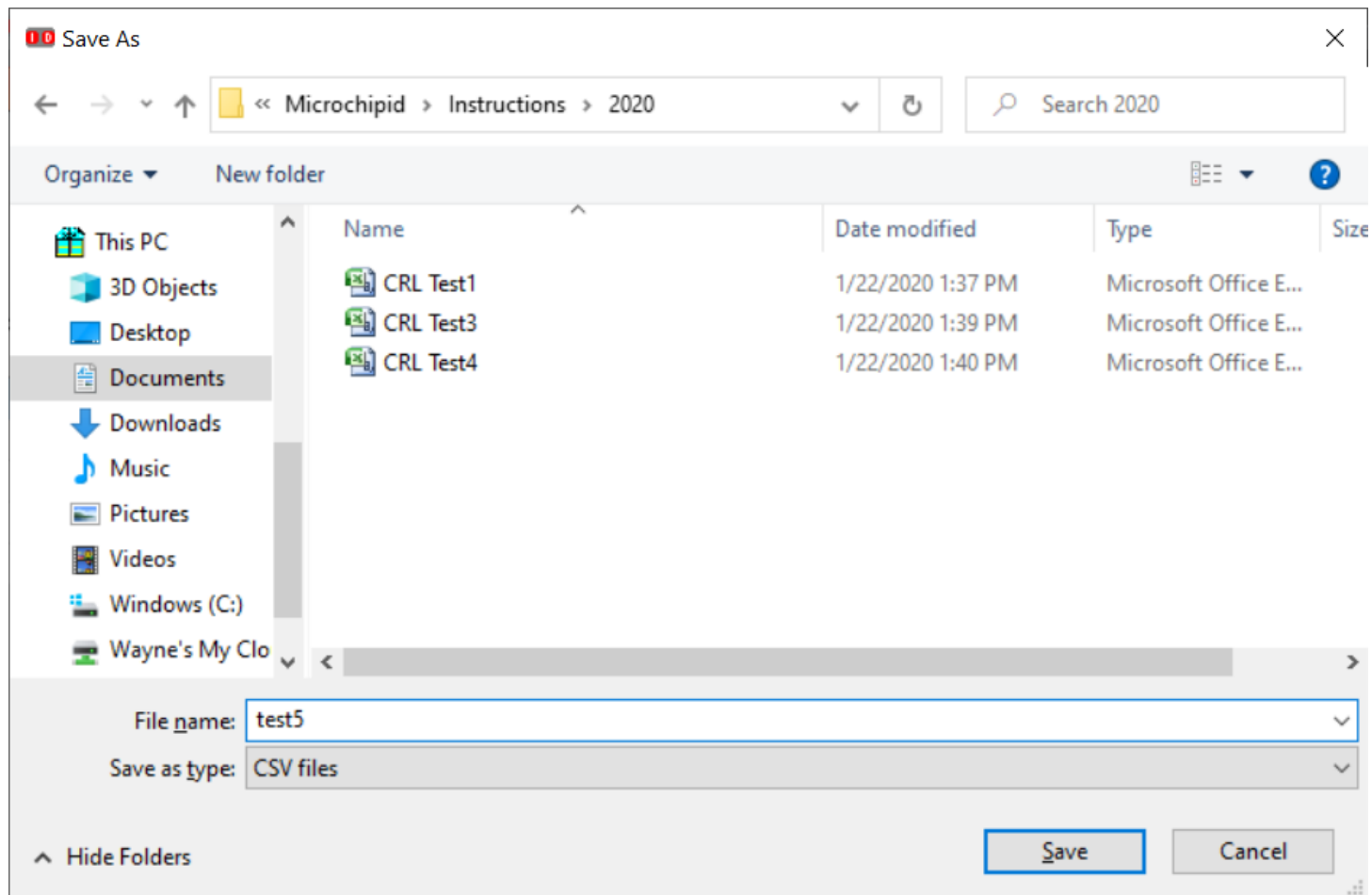

Upload to Reader

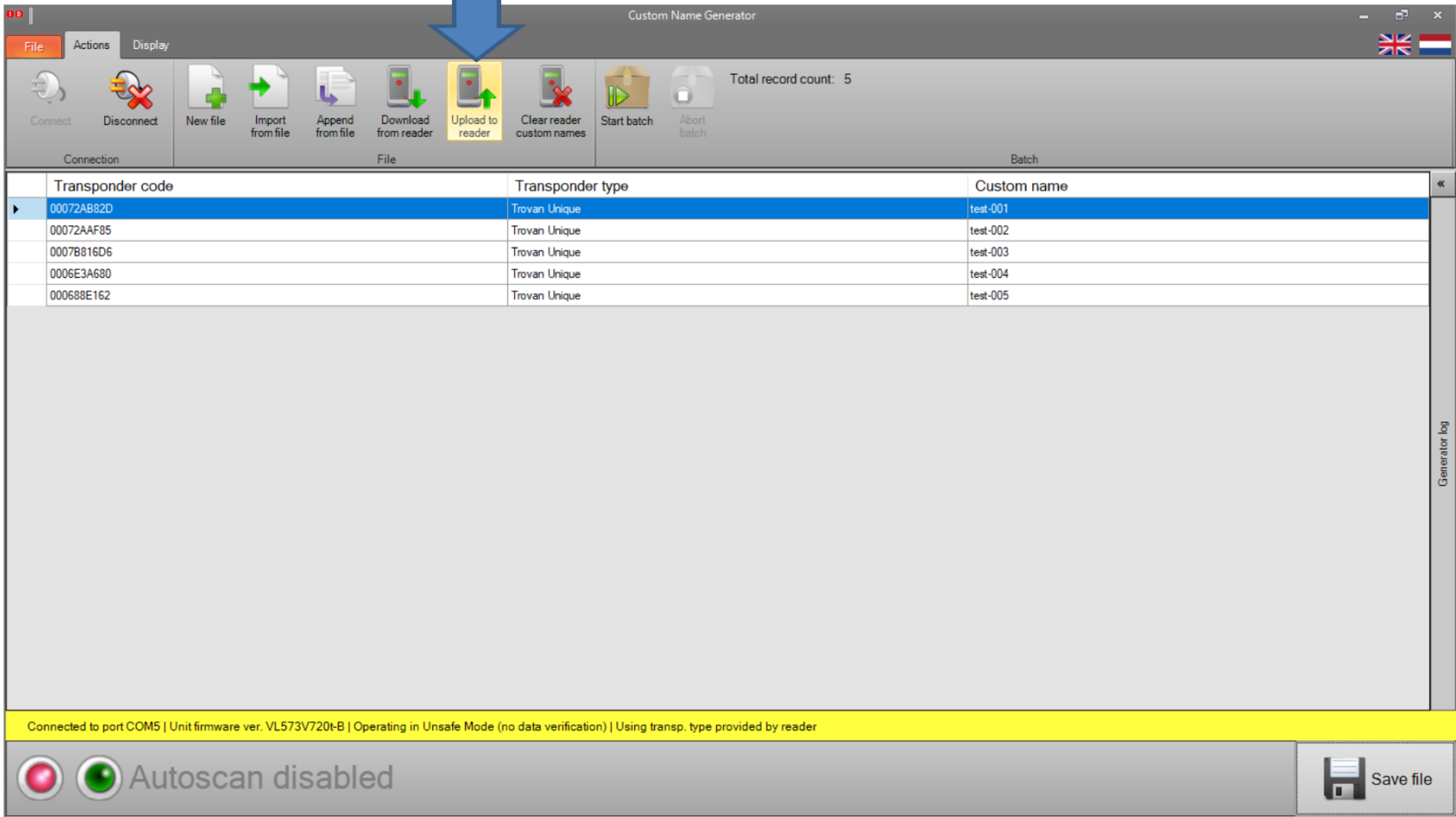

Custom Name Generator

File Actions Display

Connect Disconnect New file Import from file Append from file Download from reader Upload to reader Clear reader custom names Start batch Abort batch Total record count: 5

| Transponder code | Transponder type | Custom name |
|------------------|------------------|-------------|
| 00072AB82D | Trovan Unique | test-001 |
| 00072AAF85 | Trovan Unique | test-002 |
| 0007B816D6 | Trovan Unique | test-003 |
| 0006E3A680 | Trovan Unique | test-004 |
| 000688E162 | Trovan Unique | test-005 |

Connected to port COM5 | Unit firmware ver. VL573V720t-B | Operating in Unsafe Mode (no data verification) | Using transp. type provided by reader

Autoscan disabled Save file



Download from Reader



Custom Name Generator

File Actions Display

Connect Disconnect New file Import from file Append from file **Download from reader** Upload to reader Clear reader custom names Start batch Abort batch Total record count: 5

| Transponder code | Transponder type | Custom name |
|------------------|------------------|-------------|
| 00072AB82D | Trovan Unique | test-001 |
| 00072AAF85 | Trovan Unique | test-002 |
| 0007B816D6 | Trovan Unique | test-003 |
| 0006E3A680 | Trovan Unique | test-004 |
| 000688E162 | Trovan Unique | test-005 |


Connected to port COM5 | Unit firmware ver. VL573V720t-B | Operating in Unsafe Mode (no data verification) | Using transp. type provided by reader

Autoscan disabled Save file

Generator log




Click on Yes

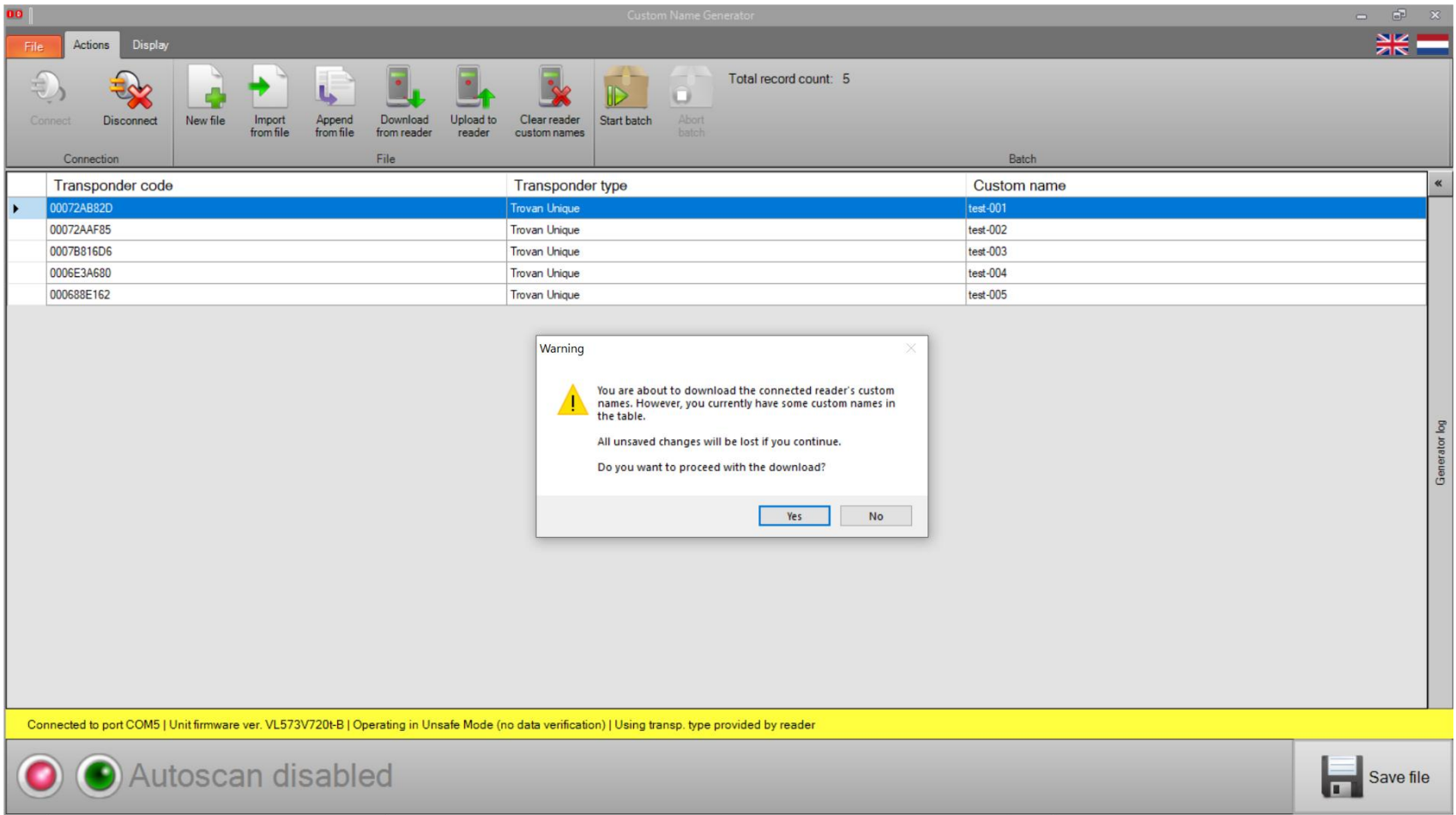

The screenshot shows the 'Custom Name Generator' application window. The interface includes a menu bar (File, Actions, Display), a toolbar with icons for Connect, Disconnect, New file, Import from file, Append from file, Download from reader, Upload to reader, Clear reader custom names, Start batch, and Abort batch. A 'Total record count: 5' is displayed. Below the toolbar is a table with three columns: Transponder code, Transponder type, and Custom name. The table contains five rows of data. A 'Warning' dialog box is open in the center, with a yellow warning icon and the following text:

Warning

 You are about to download the connected reader's custom names. However, you currently have some custom names in the table.

All unsaved changes will be lost if you continue.

Do you want to proceed with the download?


Yes No

At the bottom of the application window, a yellow status bar displays: 'Connected to port COM5 | Unit firmware ver. VL573V720t-B | Operating in Unsafe Mode (no data verification) | Using transp. type provided by reader'. Below the status bar, there are two status indicators: 'Autoscan disabled' and a 'Save file' button.

| Transponder code | Transponder type | Custom name |
|------------------|------------------|-------------|
| 00072AB82D | Trovan Unique | test-001 |
| 00072AAF85 | Trovan Unique | test-002 |
| 0007B816D6 | Trovan Unique | test-003 |
| 0006E3A680 | Trovan Unique | test-004 |
| 000688E162 | Trovan Unique | test-005 |



Erase or append to the reader

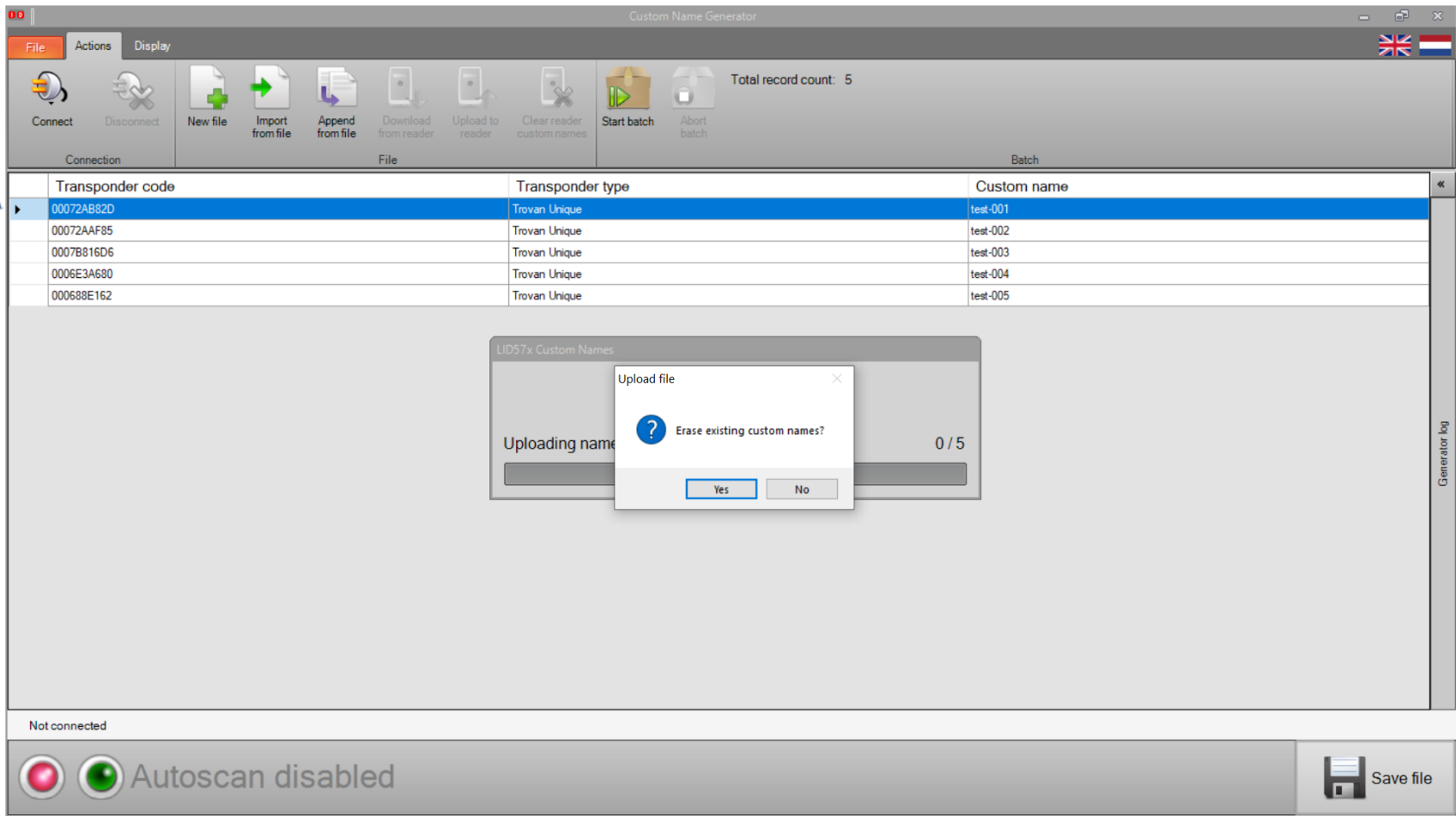

Custom Name Generator

File Actions Display

Connect Disconnect New file Import from file Append from file Download from reader Upload to reader Clear reader custom names Start batch Abort batch Total record count: 5

| Transponder code | Transponder type | Custom name |
|------------------|------------------|-------------|
| 00072AB82D | Trovan Unique | test-001 |
| 00072AAF85 | Trovan Unique | test-002 |
| 0007B816D6 | Trovan Unique | test-003 |
| 0006E3A680 | Trovan Unique | test-004 |
| 000688E162 | Trovan Unique | test-005 |

Uploading name 0 / 5

Upload file

Erase existing custom names?

Yes No

Not connected


Autoscan disabled

Save file


Generator log



Custom Name Uploader

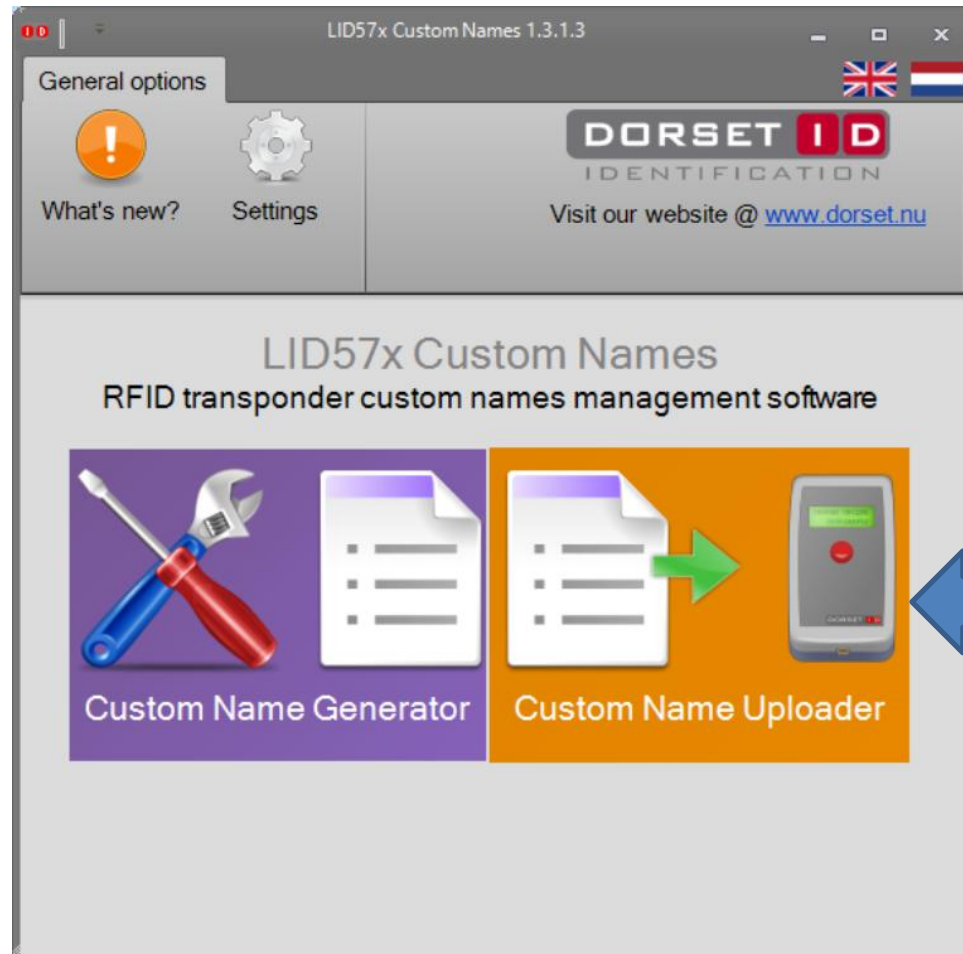

Open File

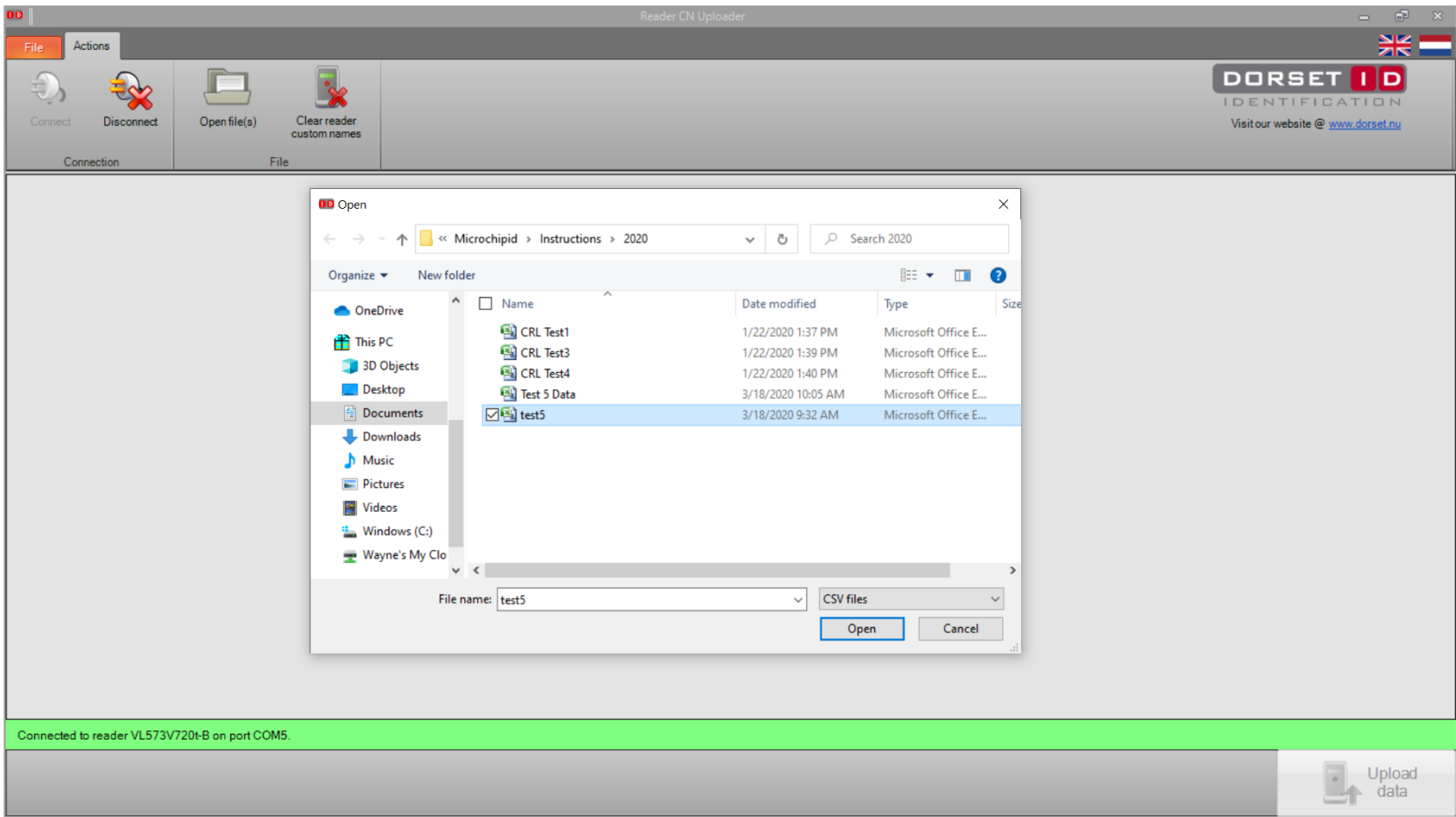

Reader CN Uploader

File Actions

Connect Disconnect Open file(s) Clear reader custom names

Connection File

DORSET ID
IDENTIFICATION
Visit our website @ www.dorset.nu

Open

< > << >> << Microchipid > Instructions > 2020 Search 2020

Organize New folder

| Name | Date modified | Type | Size |
|---|--------------------|-----------------------|------|
| CRL Test1 | 1/22/2020 1:37 PM | Microsoft Office E... | |
| CRL Test3 | 1/22/2020 1:39 PM | Microsoft Office E... | |
| CRL Test4 | 1/22/2020 1:40 PM | Microsoft Office E... | |
| Test 5 Data | 3/18/2020 10:05 AM | Microsoft Office E... | |
| <input checked="" type="checkbox"/> test5 | 3/18/2020 9:32 AM | Microsoft Office E... | |

File name: test5 CSV files


Open Cancel

Connected to reader VL573V720t-B on port COM5

Upload data



Information Uploaded from file

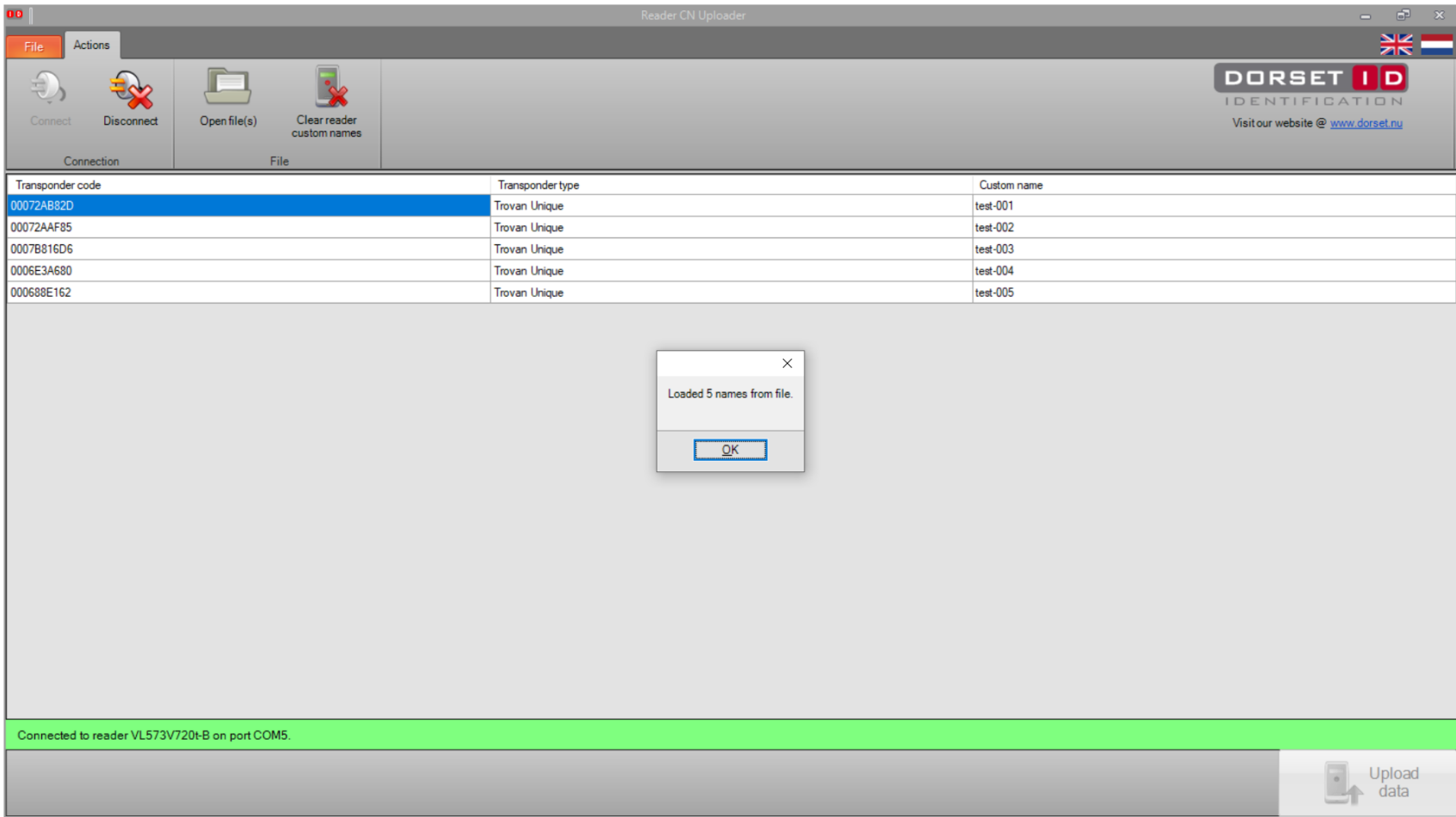

The screenshot displays the 'Reader CN Uploader' application window. The interface includes a menu bar with 'File' and 'Actions', a toolbar with 'Connect', 'Disconnect', 'Open file(s)', and 'Clear reader custom names', and a 'DORSET ID IDENTIFICATION' logo with the website 'www.dorset.nu'. A table lists transponder data, and a dialog box confirms 'Loaded 5 names from file.' A green status bar at the bottom indicates 'Connected to reader VL573V720t-B on port COM5', and an 'Upload data' button is visible in the bottom right corner.

| Transponder code | Transponder type | Custom name |
|------------------|------------------|-------------|
| 00072AB82D | Trovan Unique | test-001 |
| 00072AAF85 | Trovan Unique | test-002 |
| 0007B816D6 | Trovan Unique | test-003 |
| 0006E3A680 | Trovan Unique | test-004 |
| 000688E162 | Trovan Unique | test-005 |

Loaded 5 names from file.

Connected to reader VL573V720t-B on port COM5

Upload data



Clear reader custom names



Reader CN Uploader

File Actions

Connect Disconnect Open file(s) Clear reader custom names

Connection File

DORSET ID
IDENTIFICATION
Visit our website @ www.dorset.ru

| Transponder code | Transponder type | Custom name |
|------------------|------------------|-------------|
| 00072AB82D | Trovan Unique | test-001 |
| 00072AAF85 | Trovan Unique | test-002 |
| 0007B816D6 | Trovan Unique | test-003 |
| 0006E3A680 | Trovan Unique | test-004 |
| 000688E162 | Trovan Unique | test-005 |

Connected to reader VL573V720t-B on port COM5

Upload data



Click on Yes

The screenshot shows the 'Reader CN Uploader' application window. The interface includes a menu bar with 'File' and 'Actions', a toolbar with 'Connect', 'Disconnect', 'Open file(s)', and 'Clear reader custom names', and a table of transponder data. A dialog box titled 'Clear reader' is open, asking 'Erase existing custom names?' with 'Yes' and 'No' buttons. The 'Yes' button is highlighted with a blue border. A green status bar at the bottom indicates 'Connected to reader VL573V720t-B on port COM5'. An 'Upload data' button is visible in the bottom right corner.

| Transponder code | Transponder type | Custom name |
|------------------|------------------|-------------|
| 00072AB82D | Trovan Unique | test-001 |
| 00072AAF85 | Trovan Unique | test-002 |
| 0007B816D6 | Trovan Unique | test-003 |
| 0006E3A680 | Trovan Unique | test-004 |
| 000688E162 | Trovan Unique | test-005 |

Connected to reader VL573V720t-B on port COM5

Upload data



Click on OK

The screenshot shows the 'Reader CN Uploader' application window. The interface includes a menu bar with 'File' and 'Actions', a toolbar with 'Connect', 'Disconnect', 'Open file(s)', and 'Clear reader custom names', and a 'DORSET ID IDENTIFICATION' logo with the website 'www.dorset.nu'. A table displays transponder data, and a dialog box prompts the user to click 'OK'.

| Transponder code | Transponder type | Custom name |
|------------------|------------------|-------------|
| 00072AB82D | Trovan Unique | test-001 |
| 00072AAF85 | Trovan Unique | test-002 |
| 0007B816D6 | Trovan Unique | test-003 |
| 0006E3A680 | Trovan Unique | test-004 |
| 000688E162 | Trovan Unique | test-005 |

Names uploaded; old names cleared.

OK

Connected to reader VL573V720t-B on port COM5.

Upload data



Thank you for your time. If you have any questions please contact us at office@microchipid.us

