

LID 574 Toolbox Set Up





Welcome to the GR252, LID574, LID674 settings

This wizard will guide you through the
GR252, LID574, LID674 settings.

Using this wizard you can edit the settings
of your handheld scanner.

To continue, click Next.



< Back

Next >

Cancel

Select transponders to read

Please select the transponder types the reader should read



Current firmware version: LID574V1.1.1CR-B

Trovan unique

Trovan flex 3 blocks

PSK1

Trovan flex 5 blocks

PSK2

Trovan flex 7 blocks


To increase reading performance only select required transponder types.




< Back

Next >

Cancel

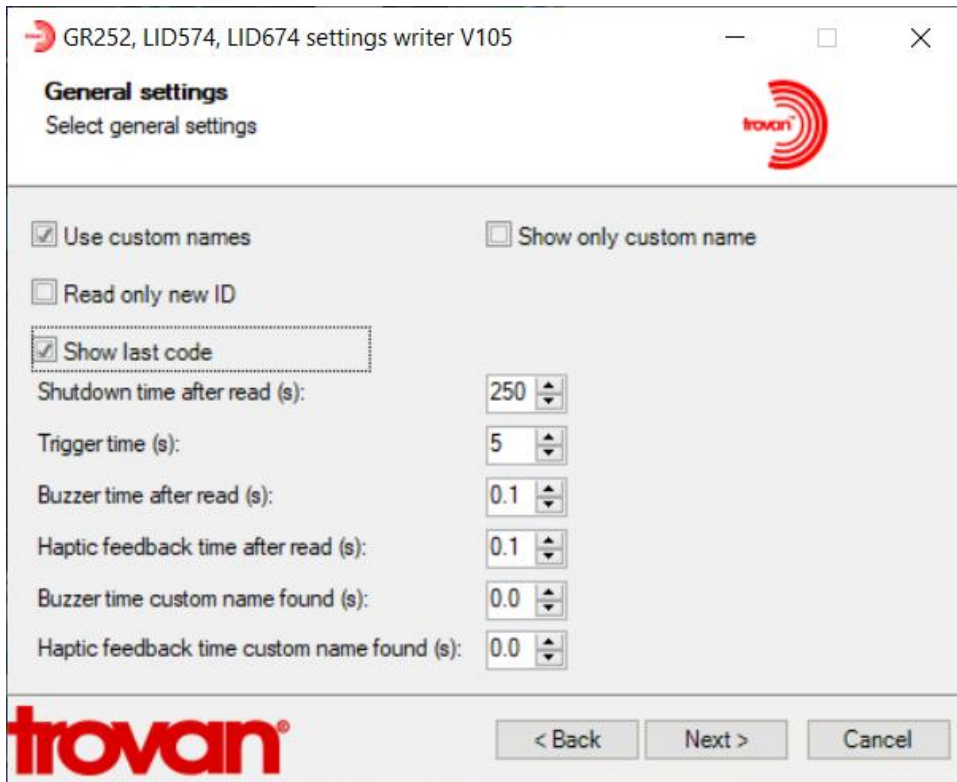

Or to reduce battery Consumption (reduce the shutdown time after read interval)

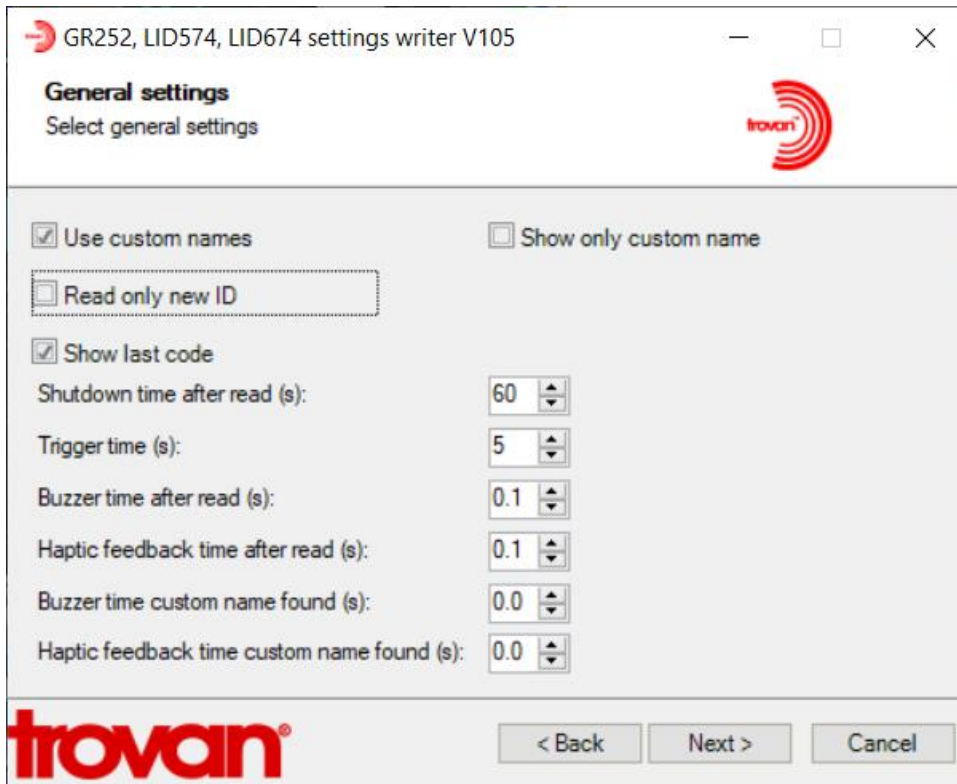

→ GR252, LID574, LID674 settings writer V105



Save settings

Save tag code to reader memory options



Save codes

with date/time

Memory size (KB) 2048

Up to number of Trovan codes without date/time: 95325

Up to number of Trovan codes with date/time: 61680

Memory used: 0 %




< Back

Next >

Cancel

Settings if you want to send the code to a spreadsheet via USB Port

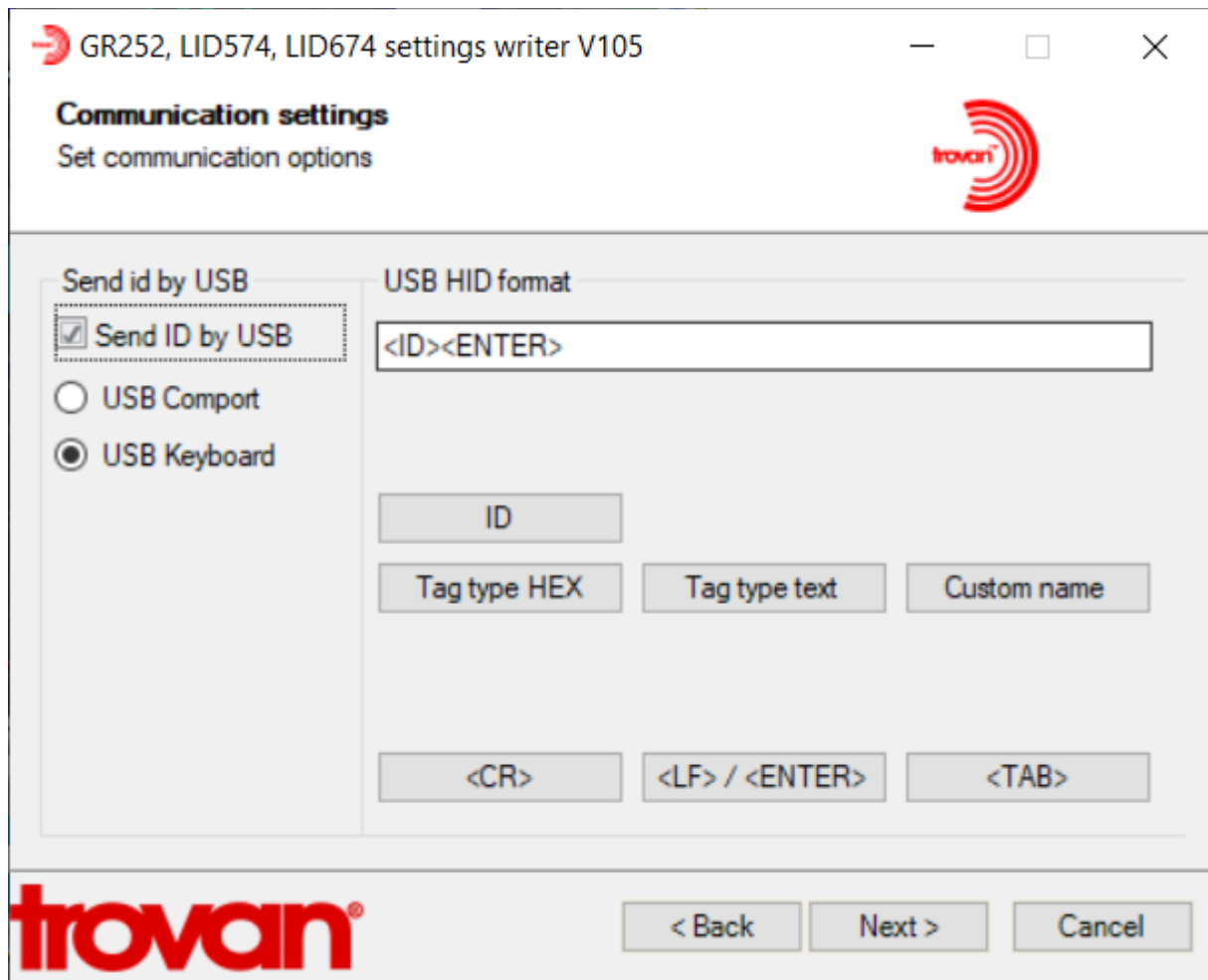

Settings if you want to send the code to a spreadsheet via Bluetooth

The screenshot shows a software window titled "GR252, LID574, LID674 settings writer V105" with a Trovan logo in the top right corner. The window is titled "Communication settings: BLE" and "Wireless connection settings".

On the left, under "Send id by Bluetooth", there are five radio button options:

- Send ID by Bluetooth
- Keep connected
- Always connected
- Bootup connect, after sent disconnect
- Connect when tag found, after sent disconnect

On the right, under "Ble format", there is a text input field containing "<ID><LF>". Below this are three buttons: "ID", "Tag type HEX", and "Tag type text". At the bottom of the "Ble format" section are three buttons: "<CR>", "<LF> / <ENTER>" (highlighted with a blue border), and "<TAB>".

At the bottom of the window, there is a Trovan logo on the left and three buttons: "< Back", "Next >", and "Cancel".

→ GR252, LID574, LID674 settings writer V105



Completed
Ready for use



The reader is ready for use.
Please disconnect the reader.



< Back

Finish

Cancel